




OS2[®]-V


VARISATION STAPLE

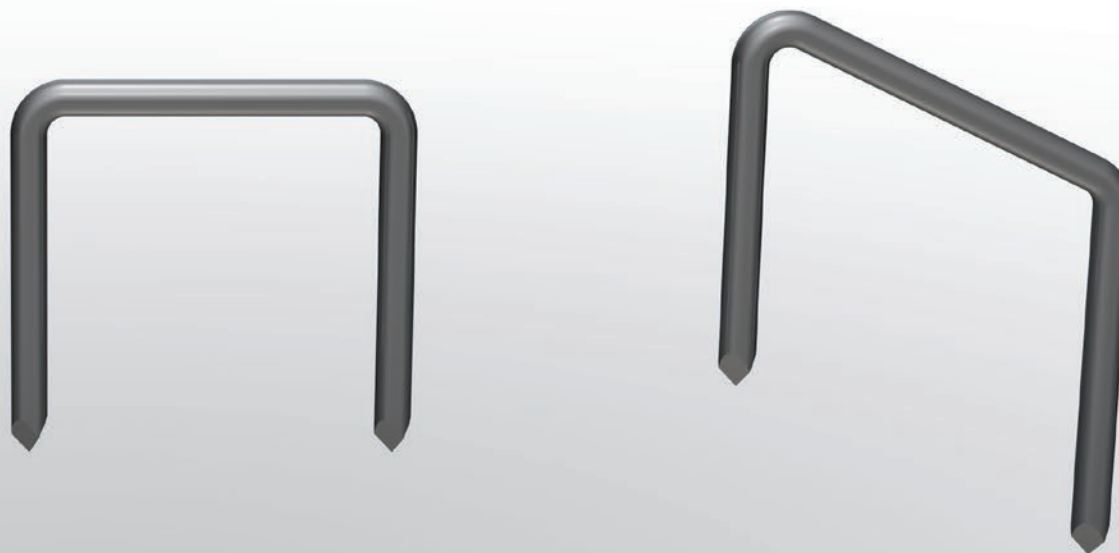


▶ PRODUCT DESCRIPTION	3
▶ INDICATIONS	5
▶ SURGICAL TECHNIQUE	6
1 - BONE PREPARING	7
2 - STAPLE INSERTION	8
▶ REFERENCES	9

OS2®-V VARISATION STAPLES

- ▶ The OS2®-V variation staple is indicated for Akin type osteotomies.
- ▶ The OS2®-V variation staple is proposed with the single use instruments Neosys®. (G02 10001)
- ▶ It is delivered sterile pre-loaded on its inserter.
- ▶ It is available in 8 or 10 mm interaxis with 2 available angles: 26° or 90°.
- ▶ Once implanted the staple is freed from its inserter by pushing the button to release the locking mechanism. ①
- ▶ The inserter can be used either as stand-alone instrument or assembled on the Neosys® Universal handle to increase the lever-arm at release.

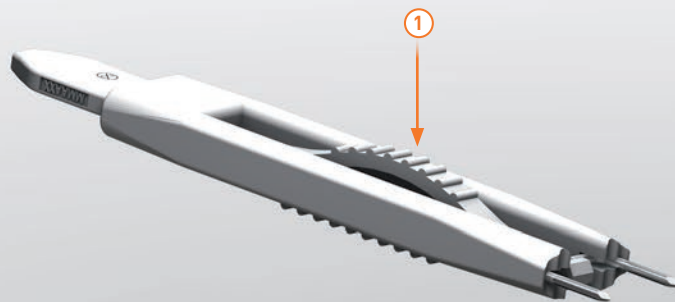

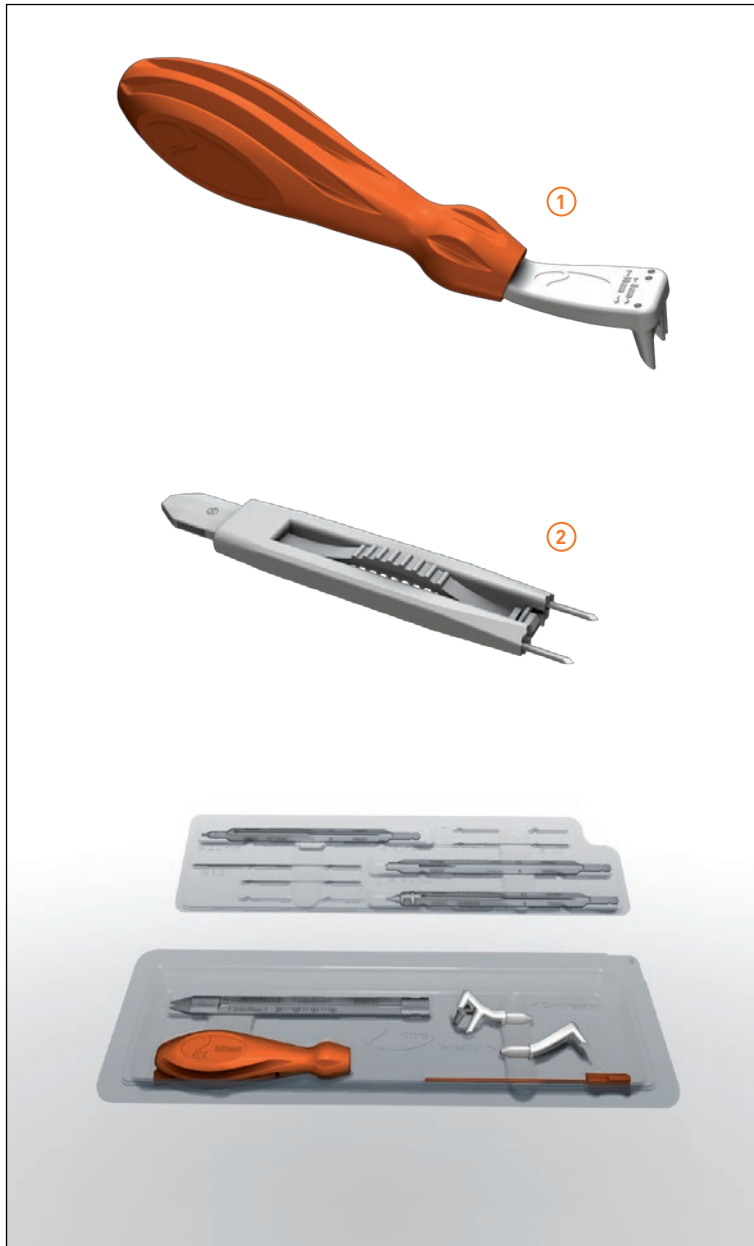

INSTRUMENTATION

› The OS2® -V variation staples are available together with the Neosys® single use instruments set.

► DRILL GUIDE ①

The OS2®-V drill guide is included in the instruments set and connects to the universal Neosys® handle.

► INSERTER ②

Adapted for each implant it allows the implantation and release of the staple by a single hand.

► IMPACTOR

Once the inserter has been detached from the staple, it can be closed and used as an impactor


INDICATIONS

- ▶ The OS2®-V Varisation staples are indicated for Akin type osteotomies.

CONTRAINDICATIONS


- ▶ The implant should not be used in a patient who has currently, or who has history of:
 - › acute or chronic systemic inflammations, whether local or systemic,
 - › active infections,
 - › sensitivity/allergies to the implant material (316L).

OS2®-V VARISATION STAPLE SURGICAL TECHNIQUE

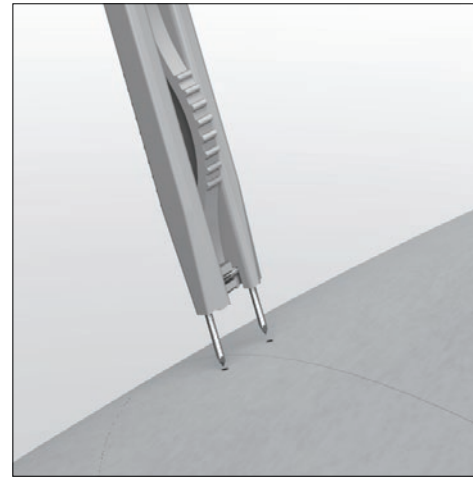


In2Bones® as the manufacturer of this device, does not practice medicine. The surgeon who performs any implant procedure is responsible for determining and using the appropriate surgical techniques for implanting the device in each patient. This Surgical Technique Manual is furnished for information purposes, as an aid to use properly the device and its dedicated instruments

1 - BONE PREPARING

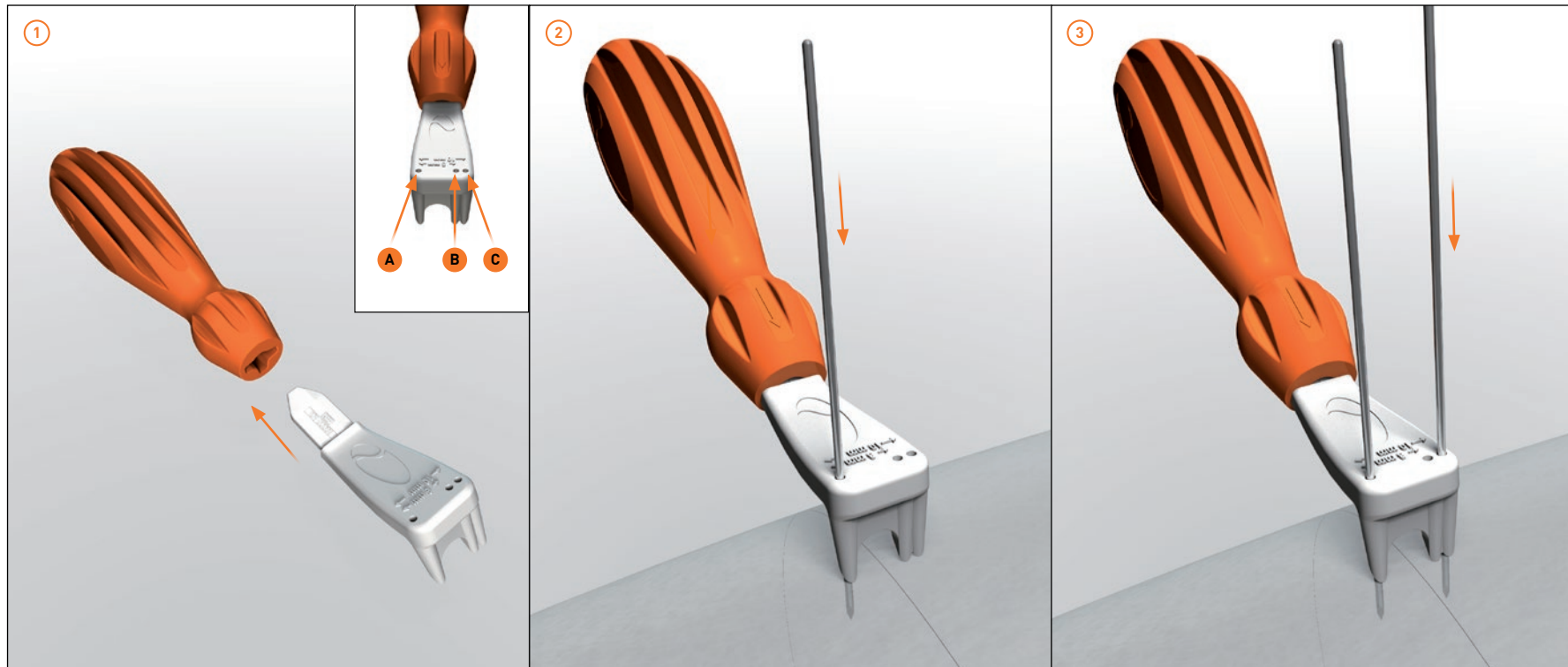


- ▶ Introduce the drill guide on the NeoSys® handle ①.
- ▶ Position the guide against the bone to select the appropriate implant size (width) and drill the leg holes A using the 1,0mm k-Wires supplied with the Neosys® set (Diam. 1.0mm, Lg 70mm) ②.
- ▶ Either hold the guide firmly and drill the second hole or leave the k-wire in situ and use a second k-wire (Diam. 1.0mm, Lg 70mm) to drill the second hole ③ at the desired width : 8mm B or 10mm C.

2 - STAPLE INSERTION

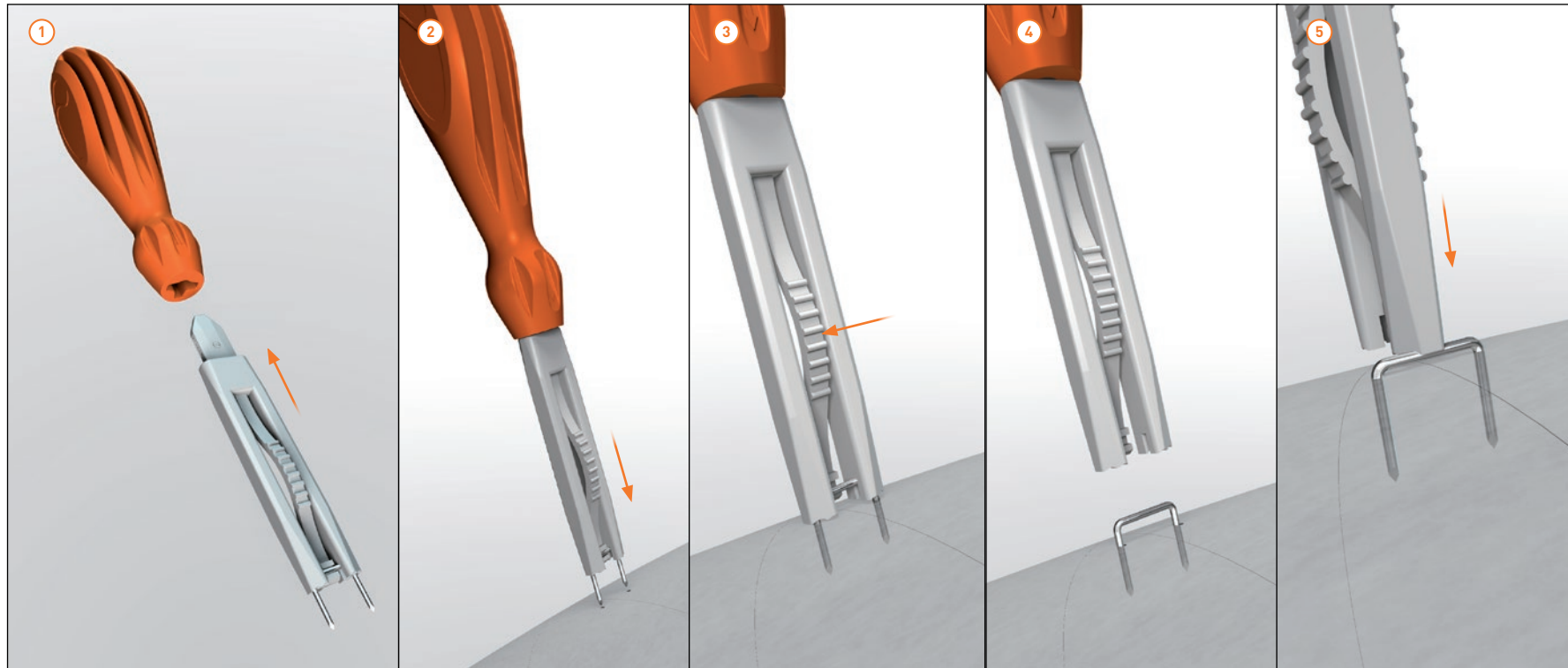

- ▶ The staple is delivered sterile and pre-loaded on its inserter.
- ▶ Either use the staple as delivered or fit the inserter on the NeoSys® handle to increase the lever arm releasing effect ①.
- ▶ Inserting the staple
 - › Position the staple legs tips in the prepared holes ②
 - › Firmly engage the staple and once fully inserted release the implant from the holder ③ ④.
 - › Impact the implant if needed. The inserter,

closed, can be used to impact the implant ⑤ or a small graft impactor can be used instead.

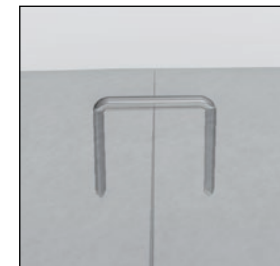

IMPLANTS

► OS2®-V

T10 SS260	OS2®-V - Varisation Staple - kit - Diam 1mm	Interaxis 10 mm	26°
T10 SS268	OS2®-V - Varisation Staple - kit - Diam 1mm	Interaxis 8 mm	26°
T10 SS900	OS2®-V - Varisation Staple - kit - Diam 1mm	Interaxis 10 mm	90°
T10 SS908	OS2®-V - Varisation Staple - kit - Diam 1mm	Interaxis 8 mm	90°

INSTRUMENTS

► Single use instrumentation

G02 10001	Neosys® kit / Osteosynthesis single use instruments
-----------	---

RECOMMANDATION

- ▶ It is recommended to carefully read the instructions for use available in the package insert.

DEVICES

- ▶ Implants :
 - › STAPLE : CE Class IIb - CE0086
- ▶ Instruments :
 - › Single use instruments: : CE Class IIa - CE0086

REIMBURSEMENT

- ▶ Reimbursement may vary from countries to countries. Check with local authorities.

MANUFACTURER

- ▶  In2Bones SAS
28, chemin du Petit Bois
69130 Ecully – FRANCE

Tél : +33 (0)4 72 29 26 26
Fax : +33 (0)4 72 29 26 29

DOCUMENT

- ▶ Reference : ST-OS2V-EN-062016-P
- ▶ Revision date: 06/2016
- ▶ Version 03



Availability of these products might vary from a given country or region to another, as a result of specific local regulatory approval or clearance requirements for sale in such country or region.

Always refer to the appropriate instructions for use for complete clinical instructions.

Non contractual document. The manufacturer reserves the right, without prior notice, to modify the products in order to improve their quality.

CAUTION: Federal law (USA) restricts this device to sale and use by, or on the order of a physician.

Except otherwise specified, all commercial trademarks used in this document, In2bones and the In2bones logo are trademarks or registered trademarks of In2bones or its subsidiaries.



www.in2bones.com

